

Avalo Caster Replacement Instructions

Tools Required

T-30 Torx Wrench 8" Tall Solid Object (optional)

Parts Required

Caster/s

Procedure

There are two ways to perform this procedure. Select from one of the following to accomplish.

Placing cart on backside

Unplug power cord and shutdown cart. Remove all drawers and items on the work surface. Secure electronics and accessories if they are not being removed.

Carefully lower the cart on its backside. This may require additional help, depending on size of cart

Remove the four screws holding the caster in place

Install new caster and reverse the procedures to complete



Lifting the side of the cart

Unplug power cord and shutdown cart.

Use caution when lifting by yourself, with an additional person, or using a jack.

Carefully lift the side of the cart and place the 8" solid object between the two casters as shown. Ensure the cart is stable before continuing. Paint can used as reference, but wood blocks or other items can be used.

Remove the four screws holding the caster in place

Install new caster and reverse the procedures to complete



**If you need any assistance during this process, please feel free to call
Capsa Healthcare Customer Service at 800-437-6633.**

